

OSL™ OverSized Litter

OSL™ The OverSized Litter: PN:60-0023 NSN:6530-01-552-7708 CE

INSTRUCTIONS FOR USE

PRODUCTS WITH A MISSION®

STEP 1:

While standing upright, release buckles from litter.



STEP 2:

Place litter on the ground and completely extend with the fabric side up.

NOTE:
When litter is in the relaxed position, the hinges closest to you (when holding the handles) will look like Figure 1.



Figure 1

EC	REP
----	-----

Mil Europe GmbH
Langerhagen Str. 71
50511 Hürth/Langerhagen
Germany

REV.022310



NORTH AMERICAN RESCUE®

www.NARescue.com • Greer, SC 29650 • 888.689.6277

OSL™ OverSized Litter

OSL™ The OverSized Litter: PN:60-0023 NSN:6530-01-552-7708 CE

(OPENING INSTRUCTIONS CONTINUED)


PRODUCTS WITH A MISSION®

● STEP 3:

Keeping the litter straight as possible, grab handles and rotate inward until all hinges rotate & lock (see Figure 2).

NOTE:

This action is best executed simultaneously using 2 individuals on each end of the litter.

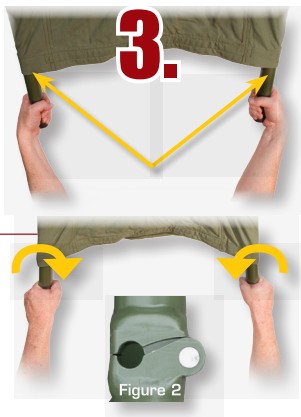

● STEP 4:

While maintaining hinges in the locked position, apply firm, steady pressure on the spreader bar with foot. Increase pressure with foot until spreader bar locks into place.


NOTE:

To close and store litter, repeat the directions in reverse order.



● STEP 5:

Place patient on litter, securing the straps around the patient, buckle and pull straps tight to avoid patient movement during transport.

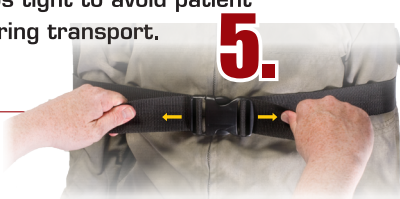

NORTH AMERICAN RESCUE®

www.NARescue.com • Greer, SC 29650 • 888.689.6277