

NAR Tactical Suction Device

PN: 10-0018 NSN: 6515-01-540-7206

QTY: 1



MDI Europa GmbH
Langenhagener Str. 71
30855 Hannover-Langenhagen
Germany

USA ONLY



(01)00842209109463 (11)

(10)



LOT

Description:

The **NAR Tactical Suction Device** consists of a rigid plastic suction tube, a suction bulb, and a collection bag for aspirated liquids. One way valves prevent the backflow of contents. Because it is manually operated, it requires little maintenance and is not dependent on batteries or other external power sources. It can be easily carried and stored, and it can be instantly made ready for use. Report any serious adverse events to the manufacturer and the competent authority.

The product or components contained in this package may constitute a medical device for which specific training is required for proper use. North American Rescue, LLC. (NAR), warrants that the product is merchantable and fit for its specified purpose. NAR expressly disclaims all other express or implied warranties relating to the product; any use beyond the product's specified purpose; or use by any party who is not trained or legally authorized to use such product. Use only as directed by your EMS authority or under the supervision of a physician.



NORTH AMERICAN RESCUE®

www.NARescue.com • 888.689.6277

35 Tedwall Court • Greer SC 29650 • USA

REV120519
PN: ZZ-0498

Intended Use:

The **NAR Tactical Suction Device** is a small, manual suction device for clearing vomitus and other foreign substances from the upper airway.

Warning:

- The NAR Tactical Suction Device should be used only by persons who have received training on suctioning the oral airway. Improper use could result in injury to the patient. Use only as directed by your EMS

authority or under the supervision of a physician. It is designed for use on adults and has not been tested to be used on pediatrics or pregnant women.

- Use of this device may result in your contact with contaminated body fluids. Use appropriate body substance isolation measures along with personal protective equipment such as gloves and eye protection.
- In the event of a Tactical Suction Device malfunction follow local protocols and report any serious incident to North American Rescue or authorized representative and the competent authority of the Member State.

Instructions for Use

- Remove the NAR Tactical Suction Device from the package. Pull the suction tube out until it stops at the red indicator.
- Clear any obvious foreign substances from the patient's airway by using finger sweeps.
- Grasp the suction bulb between your thumb and forefingers.
- Squeeze the suction bulb.
- Insert the suction tube into the back of the patient's throat. Do NOT insert farther than you can see.
- Release the suction bulb to remove the liquid from the patient's throat.
- Continue to squeeze and release the bulb to remove additional liquid.
- After the airway is cleared, it may be necessary to take other steps to clear and maintain the airway. Provide oxygen or assist ventilations as indicated in local protocols.

Caution:

Inserting the suction tube beyond visibility may restrict patient airway. Overfilling of the collection bag will result in potentially infectious fluid being released from the air vent. The Tactical Suction Device is contraindicated for pediatrics and pregnant women.

Disposal:

The NAR Tactical Suction Device is a single use device. It is designed for disposal after use. Do not attempt to clean or reuse the device, as this may increase the possibility of cross contamination. Dispose of in a manner ensuring the isolation of potential infectious substances in accordance with universal protocols.

 **WARNING:** Cancer -www.P65Warnings.ca.gov



Lot number



Consult instructions for use



Manufacturer



Do not resterilize



Do not use if package open or damaged



Date of Manufacture



Single use



Authorized Representative