

TALON® II LITTER SERIES

Talon® II 90C Collapsible Handle Litter

REF 60-0002 • NSN: 6530-01-504-9051

T3™ Lightweight Assault Litter

REF 60-0022

INSTRUCTIONS FOR USE

PRODUCTS WITH A MISSION®

- **STEP 1:**
While standing upright, release buckles from litter.



- **STEP 2:**
Place litter on the ground and completely extend with the fabric side up.

NOTE:
When litter is in the relaxed position, the hinges closest to you (when holding the handles) will look like Figure 1.



Figure 1



Relaxed Position



NORTH AMERICAN RESCUE®

www.NARescue.com • 888.689.6277

35 Tedwall Court • Greer SC 29650 • USA

TALON® II LITTER SERIES


(INSTRUCTIONS FOR USE CONTINUED)

● STEP 3:

Keeping the litter straight as possible, grab handles and rotate inward until all hinges rotate & lock (see Figure 2).

NOTE:

This action is best executed simultaneously using 2 individuals on each end of the litter.

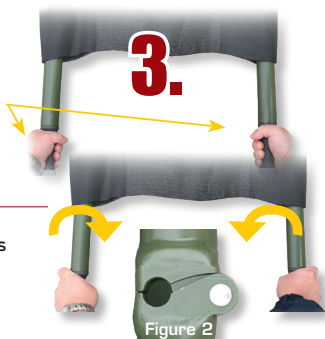

● STEP 4:

While maintaining hinges in the locked position, apply firm, steady pressure on the spreader bar with foot. Increase pressure with foot until spreader bar locks into place.

NOTE:

Apply pressure to spreader bars in a straight line parallel to the bed and poles for best results.



NOTE: To close and store litter, repeat the directions in reverse order.

● Collapsible Handle Option:

Stand upright and extend handles to full length.

NOTE:

The button will pop out when the handle is securely extended. Depress the button in order to collapse the handles.

NOTE:

To aid in deployment extend handles to full length before attempting to apply pressure to spreader bars. Ensure extendable handles are fully collapsed before collapsing spreader bars.



REF ZZ-0029

REV12092020



NORTH AMERICAN RESCUE®

www.NARescue.com • 888.689.6277

35 Tedwall Court • Greer SC 29650 • USA